100%  
MADE IN  
ITALY

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We believe that technology becomes functional when it is intuitive and efficient

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## Technology in the service of operators and patients

GIVAS presents VEGA, a revolutionary concept of electric bed, born from the synergy between our R&D Dept. and medical and nursing teams.

This partnership gave birth to a Technologic, Intuitive, Ergonomic and Efficient product that makes everyday assistance and sanitizing operations easier, thus increasing the patient's comfort and independence.

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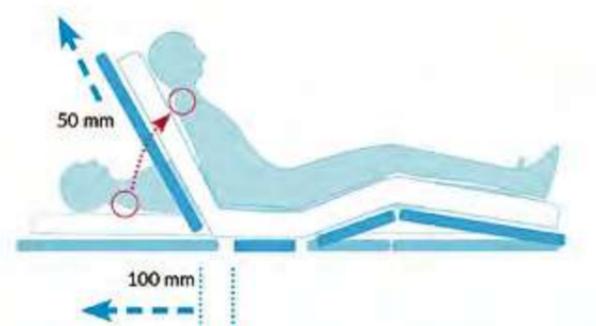
Close-up of the handles on the containment sides: they can be easily grabbed at whatever backrest incline

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## Bifunctional and self-rehabilitative ergonomomy

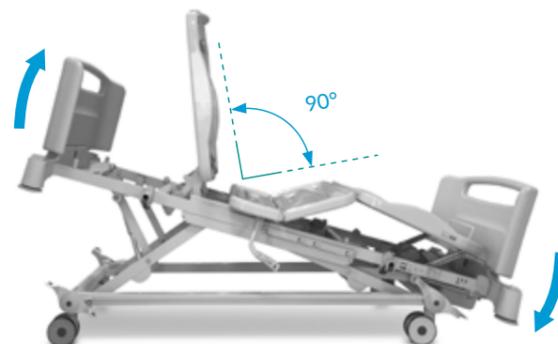
The containment sides of the Vega bed guarantee the patient's maximum protection for the entire length of the bed, as they minimize the risk of trapping limbs. (according to standard CEI EN 60601-2-52).

They are made of polypropylene, rotary printed and dyed in the mass, highly resistant even under high stress conditions. The ergonomic design of the handle allows the patient to move autonomously, thus avoiding excessive exertion for the healthcare operator while facilitating the patient's physical and psychological rehabilitation.

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"Out-of-Bed Function": bed to its minimum height, backrest up, legrest completely flat for safe getting out of bed

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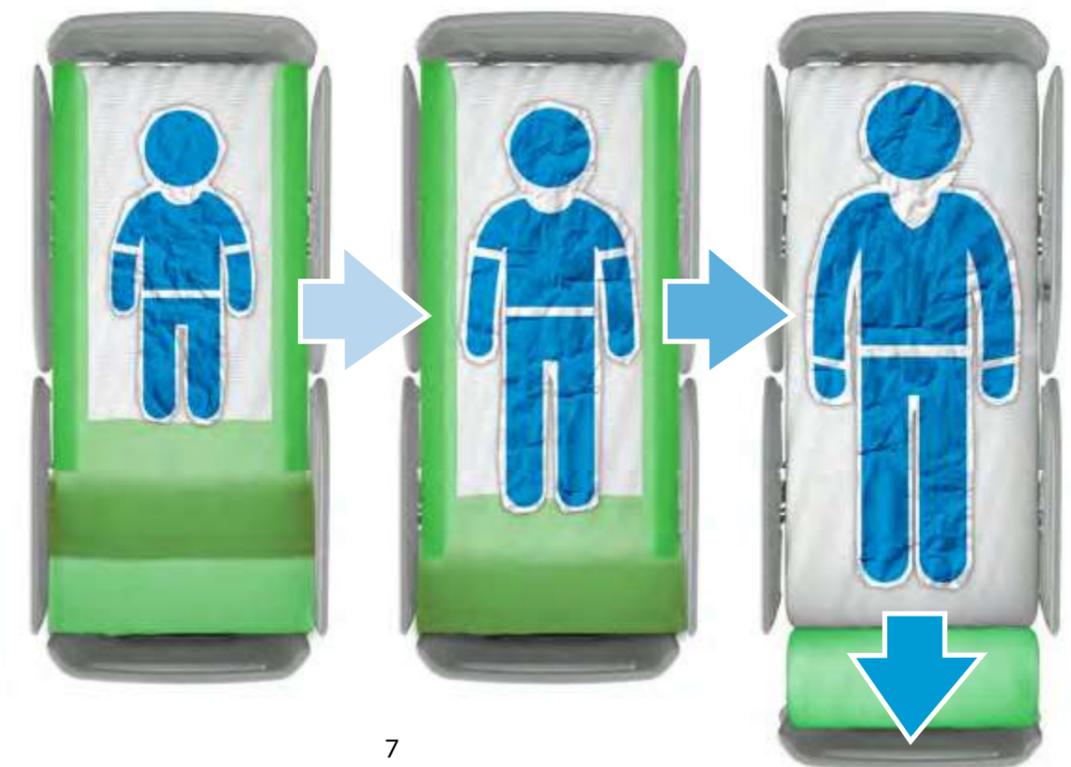
## Vega... a new generation bed

The containment area has a clean and modern design: head and foot boards curving inwards, 4-sector side-rails and non-toxic materials make the VEGA bed ideal also for pediatric use.

The VEGA bed is meant for accommodating children of different ages with their different sizes accordingly. Givas reckons that extendable pediatric beds cannot guarantee the necessary safety and comforts for patients as big as grown-ups. The VEGA bed is therefore designed for children at least 90 cm tall, thanks to special compensation cushions which reduce the lying surface, as shown below.

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VEGA: versatility and utmost comfort in order to accommodate any guest

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## Total safety in just one look

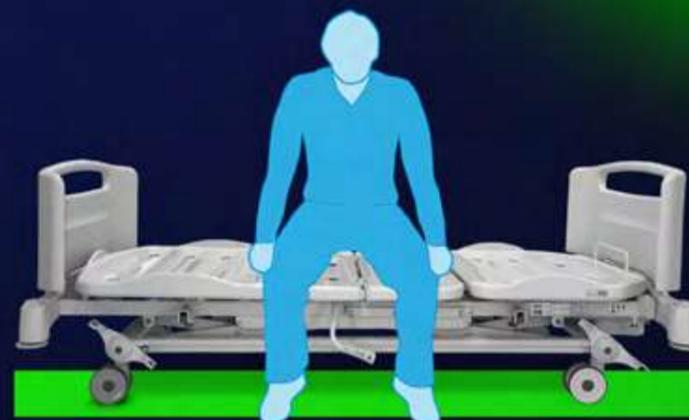
Night hours are the most delicate time of the day for healthcare operators as they must ensure total safety for the patients. Wrong height and section incline adjustments are the main causes for accidental falling of patients when getting out of the bed. Vega changes all of that: just one look is enough to know if the bed is in the safest night position. In SAFE MODE a GREEN LED light placed under the bed casts a light beam on the floor that lets the operator know if the bed is in the right position. If not, the light beam is RED.

### SAFE POSITION

VEGA casts a GREEN light beam when:

- the lying surface is completely flat
- the bed is at its minimum height

If one of these settings is not complied with, then the RED light will be automatically activated.

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
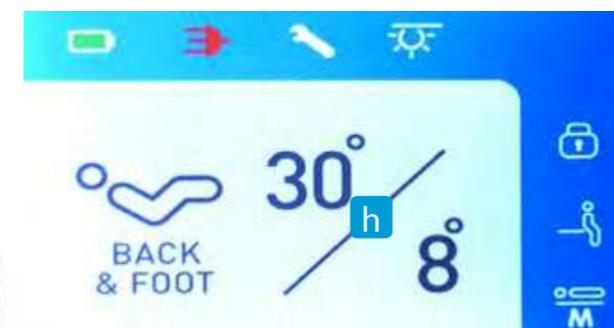
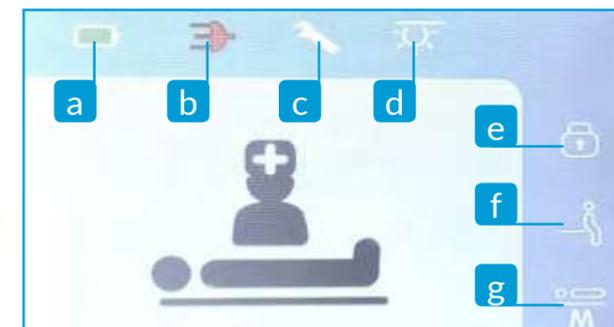
## It is simple and safe to have total control in your hand

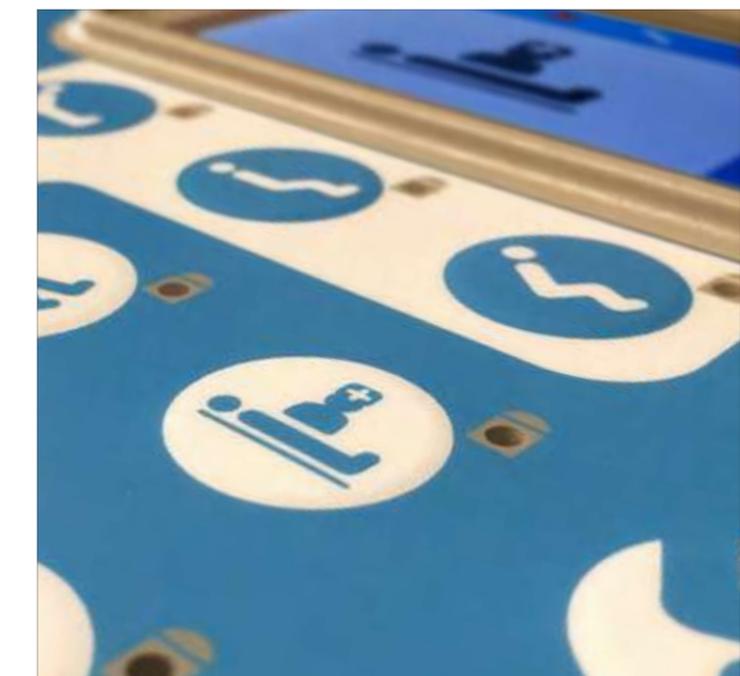POD control is much more than a simple Supervisor of bed functions and movements: it is a total monitoring system of all those bed functions that can be required by hospital nurses and healthcare operators.

Some of the most important and innovative functions are:

- a) - Battery charge indicator;
- b) - Indicator signalling when bed is connected to the power grid;
- c) - Indicators for maintenance needs;
- d) - Safety nightlight indicator when RED/GREEN light is on;
- e) - Indicator of locked functions;
- f) - Alarm when patient gets out of the bed unattended;
- g) - Scale management;
- activation of foot controls locking
- memorization of lying surfaces positions
- activation of emergency Trendelenburg
- activation of backrest quick release for CPR
- h) - Indicator of incline degrees of each section.

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(Operator Control Panel with multifunction Display)

RAT525 (Opt.)  
POD control  
POD control

RAT520 (Opt.)  
Supervisor standard  
Standard supervisor

Pedaliere abbinata a ruote:  
Foot control for wheels:  
RCTXXX

RAT505/510/515 (Opt.)  
Pannello flessibile FPP  
Flexible FPP panel

RAT500 (Opt.)  
Pulsantiera  
Hand control

Comandi integrati per:  
Integrated control for:  
VEGA EBT508-518

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Agile, light and easy to move, and above all steady, sturdy and functional

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
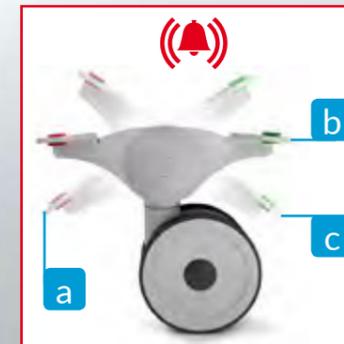
VEGA is equipped with a 4-wheel braking system with directional control: the foot controls placed at the 4 corners of the bed guarantee safety and prompt intervention. The locking system allows:

- a) - To simultaneously lock all wheels;
- b) - To simultaneously unlock all wheels;
- c) - To activate the directional lock on a single wheel, while leaving the others free; if endowed with a 5<sup>th</sup> wheel, this one will be locked.

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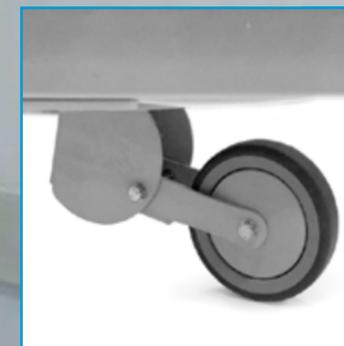
If wheels are unlocked, an alarm signals when the bed is stationary and connected to the grid

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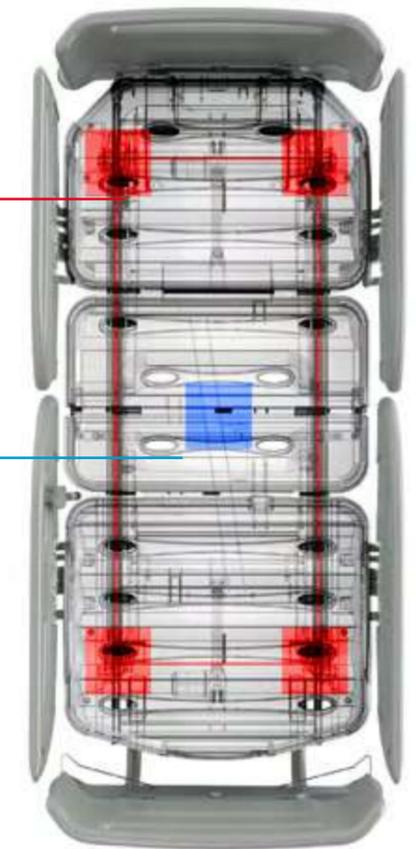The 5<sup>th</sup> wheel (Ø 125 mm) guarantees top manoeuvrability in difficult steering situations; it also allows the bed to turn 360 degrees

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VEGA can be equipped with a 5<sup>th</sup> wheel placed at the center of the bed that helps to move it and allows it to turn 360 degrees. The 5<sup>th</sup> wheel is endowed with directional lock, activated by the braking system, and with a cushion system that minimizes floor imperfections.

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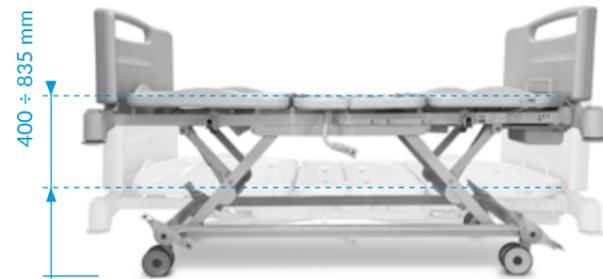
## Reliability in any situation

A large lying surface is the necessary starting point for offering top comfort to patients. The ABS surface has been carefully designed so as to be corner-free, with soft lines and ventilation ducts that guarantee proper aeration to the mattress, thus avoiding any dangerous and annoying inconvenience for the patients. An integrated system fastens the mattress in six different spots: this prevents any misalignments between the ABS surface and the mattress when the bed is moved.

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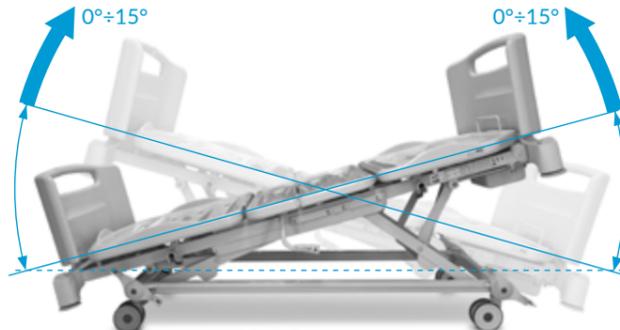
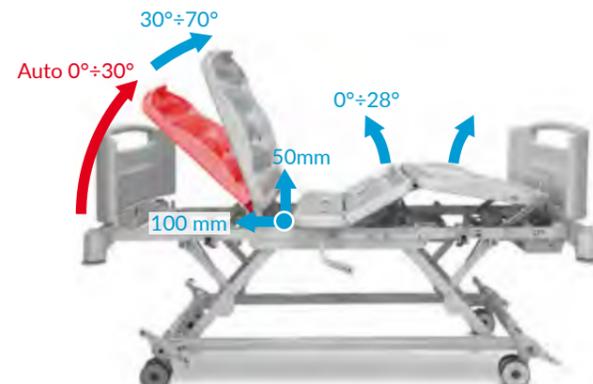
## Different-sized surfaces

- "A" - Roto-translating backrest section
- "B" - Fixed pelvis section
- "C" - Femoral section
- "D" - Legrest section with hinge endowed with negative rotation lock

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Backrest with amortized descent and raising with stop at 30° to prevent ventilation pneumonia

- 1** Nuovo design ergonomico per testiere e pediere  
New ergonomic design for head/foot boards
- 2** Innesto asta porta flebo esterna al letto  
Housing for I.V. stand outside the lying surface
- 3** Monocomando blocco/sblocco testiera/pediera  
Single control for locking/unlocking of head/foot boards
- 4** Pedaliera bilaterale a doppio impulso  
Double-pulse bilateral foot control
- 5** Comandi movimentazioni bilaterali interni/esterni integrati nella sponda  
Inside/outside bilateral movement controls, integrated on the side-rails
- 6** POD control (Pannello Operatore con Display multifunzionale)  
POD control (Operator Control Panel with multifunction Display)
- 7** Supervisor standard  
Standard supervisor
- 8** Pannello pensile flessibile FPP  
Flexible hanging FPP panel
- 9** Pulsantiera con supporto (RAT620 Opt.)  
Hand control with support (RAT620 Opt.)

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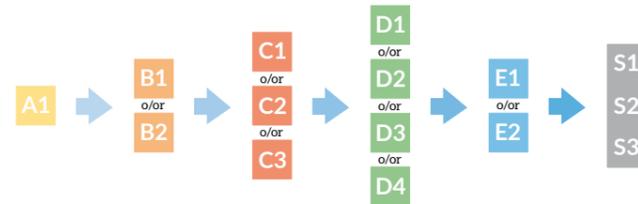
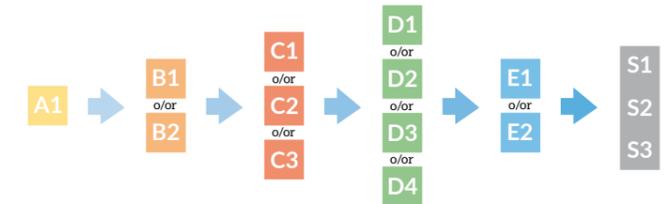
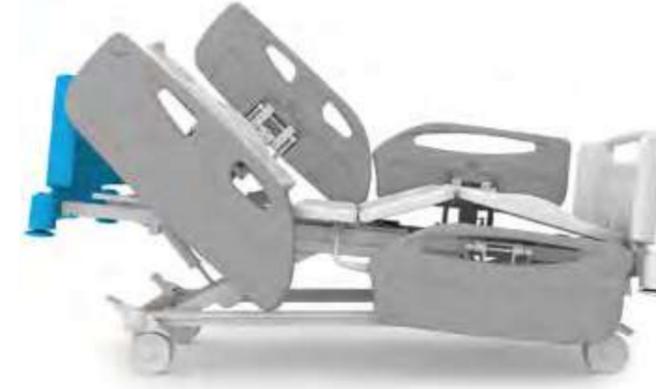
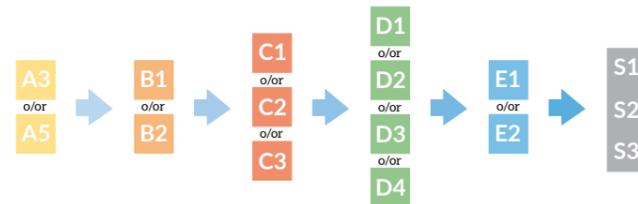
The excellence in safety  
and comfort

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## Flexibility and ergonomics for any use

Thanks to the varnish with epoxy powders and plastic coverings mixed with "Interpone®" additive, the VEGA bed is highly protected against bacteria.

Its surfaces are seamless and without fixing screws, therefore they can be quickly and efficiently sanitized.

Furthermore, it is equipped with big and sliding wheels and with a balanced structure that allow the operator to push the bed with less effort than usual.

It is also possible to add a 5<sup>th</sup> optional wheel that increases its manoeuvrability even with sharp bends.

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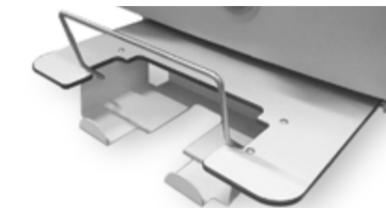
RA0112  
Portamonitor  
Monitor support

RAT610  
Portacoperte/Porta ACO  
Blanket support/ACO support

RA0148  
Portabombola  
Oxygen tank holder

RA0023  
Asta flebo regolabile  
a 2 ganci  
Adjustable I.V. stand,  
2 hooks

RA0043  
Asta flebo regolabile  
a 4 ganci  
Adjustable I.V. stand,  
4 hooks

RA0050  
Asta solleva malati  
standard  
Standard lifting  
pole

RA0053  
Asta solleva malati  
regolabile con pulsante  
Adjustable lifting pole  
by push-button

RA1050 - Supporto bottiglia / Bottle support  
RA1055 - Canule / Straws

RAT620  
Porta pulsantiera  
Hand control support

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AQUAMIKE is a simple aid which allows patients to be autonomous by enabling them to have a drink at any time without using their hands or requiring help from carers, thus avoiding all movements that might be impossible or unwise.

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Made of polyurethane foam of the latest generation, without chlorofluorocarbons. **(A1)** total dips of 30mm, hollow -15 mm, peak +15 mm, self-massaging; it fits perfectly to beds with articulated joints. The coating is made of an inherently flame-retardant fabric that does not lose its own features of fire-retardant while washing. Equipped with a zipper along three sides for maintenance operations.

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Made of molecular expanded polyurethane with open cells that guarantees maximum air permeability. Differentiated lying surfaces and coatings: **(A1)** total dips of 30mm, hollow -15 mm, peak +15 mm, self-massaging; **(A2)** completely smooth; **(B1)** fireproof, water-repellent and antibacterial coated polyurethane, **(B2)** FR polyester washable at 90°C.

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Double-layered mattress and 2 differentiated coatings. **(C1)** Viscoelastic polyurethane foam (Memory Foam), suitable for patients at risk for decubitus ulcers. **(C2)** water foam layer with open cells, high elastic response in order to give maximum comfort and ideal posture also to medium/long-stay patients. **(B1)** fireproof, water-repellent and antibacterial coated polyurethane, **(B2)** FR polyester washable at 90°C.

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Mattress with 20 interchangeable parallel elements, which may be inflated in static or alternate way thanks to a digitally controlled compressor. Designed for treatment/prevention of decubitus ulcers in hospital. Equipped with fireproof bielastic PU covering and with C.P.R. valve for immediate deflating. Maximum load: 200 kg.

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## Dynamic graphics

Graphics printed directly on PVC, for VEGA head/foot boards and 4-sector side-rails; the "Dynamic" version is available in all the colors below. It is furthermore possible to customize graphics according to the furniture style of the final location. Available upon request, minimum quantity: 10 pieces.

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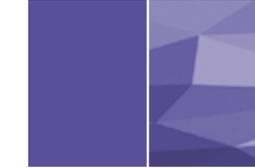
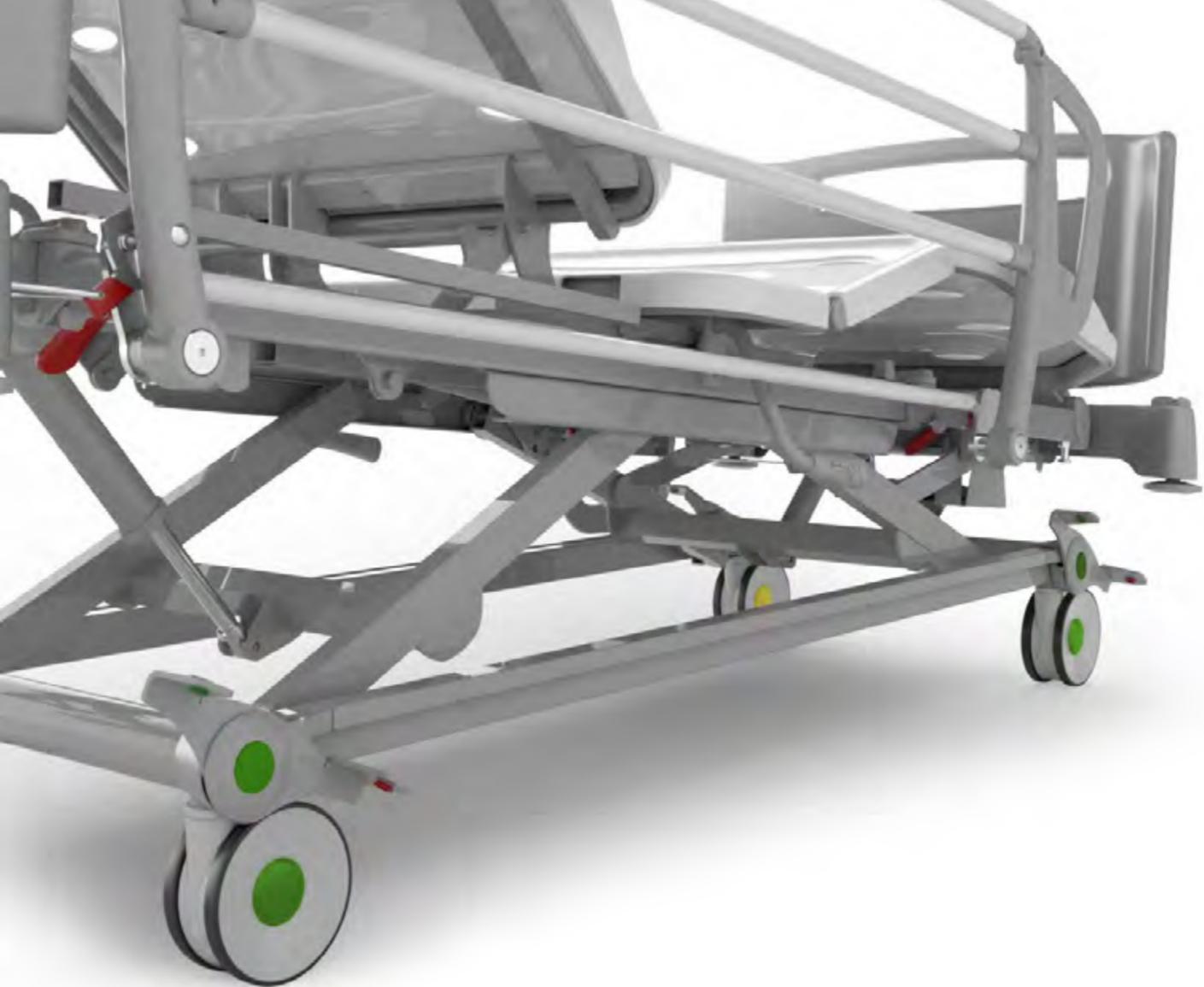
Stickers set for VEGA head/foot boards

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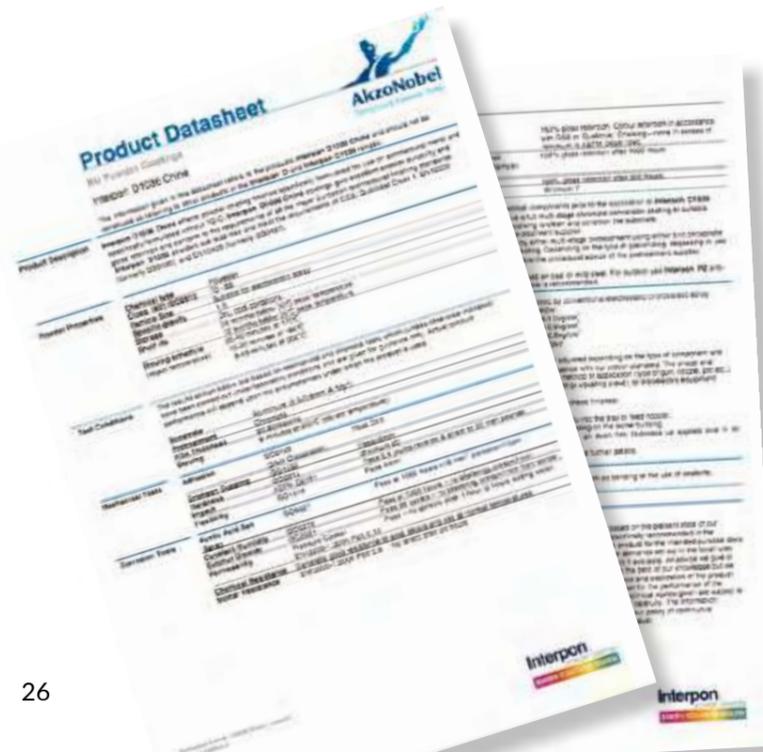
The peculiar essential design of base and braking levers has been specially conceived in order to reduce as much as possible the areas where microbes may linger. All steel parts varnished with epoxy powders, as well as coverings and plastic elements are mixed with "Interpon" additive which guarantees high quality protection against bacteria attacks.

The total absence of Volatile Organic Compounds (VOCs) and heavy metals is by far the main advantage of powder varnishes, compared to the alternatives available on the market.

Furthermore, the particular composition of "Interpon" reduces the risk of fires and gives the possibility of recycling over 95% of the waste powder.

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**GIVAS S.r.l.**